

*fistula*STOP<sup>®</sup>

**An innovative  
fistula pressure aid**



# FistulaStop replaces manual-pressure after dialysis treatment

## FistulaStop is comfortable for the patient

- Gentle towards fistulas and grafts
- Applies an even pressure every time
- An easy way to stop the blood flow
- Replaces need for patient to press on fistula following dialysis

## FistulaStop provides benefits for the health care workers

- Ergonomic as it replaces pressing by hand
- Easy to clean with alcohol or hot water
- Re-usable
- Can easily be assigned to a particular patient through name-tagged storage box
- One size fits all
- Time saving

## FistulaStop is easy to use

- Place a gauze swab between the access of the fistula and the FistulaStop
- Be sure that the bulge covers the fistula access
- Stretch the strap around the arm and secure it
- Remove FistulaStop when the blood flow has stopped and before the patient goes home
- FistulaStop should be changed once a month (if continuously used)

## FistulaStop is made in Sweden





*fistula*STOP®

#### Order information

Product	Order code
FistulaStop 1 pair	F 500

CE

FistulaStop is a patient bound product. It is delivered in pairs in a plastic box with a name tag attached on it.

Design protected reg.no. CDR 001614173



## *fistula*STOP®

**An innovative fistula pressure aid**

**CureMed Nordic AB** is a Swedish medtech company with head office and warehouse in Gothenburg. In our range, you will find world-leading products within healthcare and medical technology, many of which can be customized in production. Our goal is to always offer innovative products from leading manufacturers in a cost-effective manner to the healthcare sector.

For more information contact us at [info@curemednordic.se](mailto:info@curemednordic.se) or phone +46 31 47 58 00

[www.curemednordic.se](http://www.curemednordic.se)

WWW. [curemednordic.se](http://curemednordic.se)